

# Medtrica Solutions, Ltd



## **Appli-Kit Concentrate Bedside Cleaning Kit**

### **Directions for Use**

**ATC500070 / AFC500100**

#### **Directions for Use:**

1. *Open package using the tear notches*
2. *Add up to 500 ml of water*
3. *Squeeze the sponge 5 – 7 times to activate the enzymatic detergent concentrate*
4. *Wipe down the exterior of the endoscope with the sponge*
5. *Place the distal end of the endoscope into the pouch and cycle the enzymatic detergent solution through the biopsy channel*
6. *Sponges can be used to further pre-clean the scope in the sterilization area.*
7. *Discard the sponge, bag and fluid using approved methods for materials that have been in contact with possible infectious waste.*

#### **Product Details:**

- *For precleaning of instruments after a procedure. Low foaming formula*
- *Enzymatic Detergent concentrate contains 3 enzymes - Protease, Amylase and Lipase, PLUS Surfactant Detergents to break down proteinaceous bioburden during the preclean of surgical instruments*
- *Tube Sponge size: 4.25"L x 2.25"D w/ 0.25" center hole (105mm x 57mm w/ 6mm center hole)*
- *Flat Sponge size: 5"L x 4"W x .5H (127mm x 102mm x 12.5mm)*
- *Green Friendly, High Density, 100% Medical Grade foam, No Flaking or Scratching*
- *Biodegradable, non-toxic, odorless and non-irritating to the eyes*
- *Notched, easy tear open packaging*

*Options: Tubular or Flat Sponge*