



Medtrica
4001 Inglewood Ave, 101-128
Redondo Beach, CA 90278
P: 877-703-1079/F: 310-263-9098
www.medtrica.com

Dartex™ 4 way Stretch Fabric Cleaning Instructions

Directions:

Disinfection: Most phenolic or quaternary type disinfectants can be used on Polyurethane coated fabric.

User must dilute disinfectants and germicides in accordance with manufacturer's instructions.

CORRECT DILUTIONS OF THE CLEANING AGENT AND PROPER RINSING AFTER CLEANING IS CRITICAL.

Blood Contamination: Contamination with blood on Polyurethane coated fabric can be disinfected with 1:10 dilution of standard bleach (5.25% sodium hypochlorite).

*Warning: **prolonged** contact with disinfectants and cleaning agents, improper dilution ratios for any cleaning agent, or poor rinsing of fabric surfaces after disinfecting and cleaning may cause damage to both the appearance and performance of this fabric.*

Warning: Do not use harsh cleaners or solvents.

Accelerated hydrogen peroxide type cleaners can shorten the lifespan of the mattress cover - use with caution.

Laundering is not recommended.