

Medtrica Solutions, Ltd

4001 Inglewood Ave; 101-128

Redondo Beach, CA 90278

P: 877-703-1079/F: 310-263-9098

www.medtrica.com



DuraSponge™ retains up to 15% more cleaning solution than standard sponges or paper based wipes. It is compatible with all types of enzymatic and disinfecting solutions.

The DuraSponge™ allows for the safe cleaning of sensitive instruments and fixtures, as well as all general surfaces – it will not flake or scratch like lower quality sponge, mesh or paper based products.

Flat or Tubular Sponge

Recommended usage: 1 sponge per instrument.

Tube Sponge – Dry – (Tubular DuraSponge)

- *Smaller diameter hole than other tube sponges on the market:*
 1. *allows for tighter fit around all types of scopes when cleaning / more surface area in contact with insertion tube*
 2. *has less of a chance of falling off if used as a 'bumper guard' to protect the distal end of the scope*
- *Larger sponge / wider and longer size compared to other tube sponges – provides greater barrier protection from debris, larger surface area contacted, holds more fluid.*
- *100% medical grade, Green Friendly foam – see Green Friendly Foam*
- *Best used at bed side and or in SPD during full cleaning to wipe down insertion tube before cycling enzymatic and detergent through the scope.*
- *Tubular DuraSponge can also be used as a tip protector/bumper guard during storage and transport***
- *Available bulk – 100 per case; and Dispenser Box – 4 boxes of 25 / 100 units per case*

*** When using the tubular DuraSponge as a tip protector / bumper guard during storage it is recommended to:*

Always allow for airflow and unobstructed drainage around the distal end of the scope by keeping the end of the tubular DuraSponge even with the distal end of the scope / not "hiding" the distal end in the middle of the sponge.

Flat Sponge – Dry – (Flat DuraSponge)

- *Can use easily on all types of surfaces and instruments – conforms to all shapes*
- *100% medical grade, Green Friendly foam – see Green Friendly Foam*
- *Best used at bed side to wipe down insertion tube before cycling enzymatic and detergent through the scope and then during full clean in SPD; or for cleaning most other surfaces and instruments in a facility.*
- *Available bulk – 200 per case; and Dispenser Box – 4 boxes of 50 / 200 units per case*