

Medtrica Solutions Ltd
EndoSock™ Scope Sleeve



Directions for Use

1. After the scope has been reprocessed, slide one of the open ends of the EndoSock™ Scope Sleeve onto the insertion tube and pull the 4" open flap all the way up the tube and onto the boot of the scope.
2. Allow the EndoSock™ Scope Sleeve to cover the entire length of the insertion tube, (sleeve can be cut to length if needed or "scrunched up" on the insertion tube).
3. A stick-on I.D. label showing the date, time etc of the last reprocessing can be placed on the EndoSock™ and then easily transferred onto a patient's chart.
4. Remove and discard EndoSock™ before procedure.



For use during storage and is designed for one-time usage

EndoSock™ Scope Sleeve:

- EPS369502 - Small / 36" Long - 200 count
- EPS789501 - Large / 78" Long - 100 count