

DuraSponge™ Dry

ESD904101; ESD904104; ESD905201; ESD905204



Instructions for Use

For precleaning of instruments after a procedure

Remove a DuraSponge™ from the box

Soak a DuraSponge™ in an enzymatic detergent solution; detergent solution or water – (follow scope manufacturers recommendations re the liquid) and wipe the instrument or surface with suitable pressure to remove all visible bioburden

Special Warnings and Cautions

Designed for one-time usage

Be sure to consult the IFU of the item manufacturer to ensure full compliance with their instructions.

Treat the used DuraSponge™ as biohazardous and discard in compliance with your facility's guidelines.

Description:

- High Density, 100% Medical Grade polyurethane foam, No Flaking or Scratching. Clean, Non-sterile, NOT made with natural rubber latex

Specifications:

Store at room temperature away from sunlight.

5 year shelf life

Tube Sponge size: 4.25"L x 2.25"D w/ 0.5" center hole (105mm x 57mm w/ 12.5mm center hole)

- ESD904101 Bulk Packaged - 100 ct / 1 case - (1 bag of 100) or Dispenser Box packaged – 4 boxes of 25 ea; 100ct / 1 case

Flat Sponge size: 5"L x 4"W x .5H (127mm x 102mm x 12.5mm)

- ESD905201 Bulk Packaged - 200 ct / 1 case - (2 bags of 100) or Dispenser Box packaged – 4 boxes of 50 ea; 200ct / 1 case

Manufactured by Medtrica Solutions Ltd